

Instructions of Use;

871 CaluGloves® Medical Basic multi color nitrile disposable gloves. Sizes S, M, L, XL

The EU declaration of conformity (personal protective equipment) and the most recent version of the user manual can be downloaded from: www.carellurvink.nl (search for "871").

A. Use:

These gloves are manufactured with the utmost care and **meant to protect the hands against minimal risks only** (for example; contact with cleaning materials of weak action or prolonged contact with water).

They comply with Regulation (EU) 2016/425 and are classified in category 1 and marked accordingly.

CLL B.V. is not liable in the event of improper use of the product.

Assess the residual risks present to determine whether the gloves are suitable for their use.

Attention; replace the gloves at least once every two hours.

B. Precautions for use:

1. Always check the gloves for errors or imperfections before use. If the glove shows cracks or holes before or during use, discard them immediately.

In case of doubt; discard them and put on new gloves.

2. Never wear gloves that are dirty on the inside or in combination with dirty hands, this causes irritation and can cause skin rashes.

Only wear gloves on dry and clean hands.

3. DO NOT use these gloves while working with chemicals and / or in situations with microbiological hazards.

4. DO NOT use these gloves as protection against mechanical hazards, heat, cold or against ionizing radiation.

5. These gloves are designed for single use, you must replace them regularly.

C. Composition / allergies:

Some gloves may contain substances that can cause allergies to people who are particularly susceptible to this, resulting in irritations and / or contact allergic reactions.

If suspecting allergy; contact your doctor or dermatologist. Please be aware; this product may contain (traces of) natural rubber / latex which can cause allergic reactions; consult the technical department of CLL B.V. if you require more information.

D. Storage:

1. Store in original packaging in a cool and dry place, avoid exposure to excessive heat (>40°C) and / or moisture.

2. Keep away from sources of ozone, heat and open fire.

3. Open box should be shielded for exposure to direct sun or fluorescent lighting.

4. Do not use these gloves after the expiry date (see the reverse of the package).



E. Disposal (waste):

During use, the gloves can become contaminated with contaminants or other materials. Reuse (after cleaning) is not possible.

Respect the local regulations during storage and processing of disposed gloves.

F. Guarantee and warranty limitation:

CLL B.V. guarantees the conformity of this product with the technical standard data of CLL B.V. on the date of delivery to the customers. Except insofar as prohibited by law, the present warranty is delivered to replace any other warranty, including any warranties for suitability for a particular use; the responsibility of CLL B.V. is limited to reimbursing the costs price of the product in question. The buyers and users of the product accept these warranty conditions, which cannot be changed by any other agreement, both orally and in writing.

G. Explanation of the icons:



CE:

Category 1 (CE I)

Sizes:

The sizes do not meet the minimum length requirements according to EN 420: 2003 + A1: 2009 to increase comfort for special purposes (eg fine assembly work). Wear the gloves only in a suitable size. Gloves that are too loose or too tight restrict movement and do not provide the optimal level of protection.

H. More information:

Do you need help to determine which glove is most suitable for your task? Ask our specialists via info@carellurvink.nl or call via **0031 (0)53 - 434 4343**.

Are you curious about the other variants of our certified work gloves or our complete range of products? Go to our webshop via www.carellurvink.nl or visit one of our servicecenters in Enschede (NL), Hilversum (NL) or Winschoten (NL).

Everyone deserves a clean & safe environment

