



CAREL LURVINK

CALUGLOVES®

Instructions of Use;

896M CaluGloves® Medical True Blue nitrile disposable gloves. Sizes S, M, L, XL

The EU declaration of conformity (personal protective equipment) and the most recent version of the user manual can be downloaded from: www.carellurvink.nl (search for "896M").

A. Use: These gloves are manufactured with the utmost care and meant to protect the hands. They are liquid-tight and can therefore be used as protection against risks of a microbiological nature, they also offer (limited) protection against chemicals (detailed information is provided later in this manual). They comply with Regulation (EU) 2016/425 and are marked accordingly. CLL B.V. is not liable in the event of improper use of the product. Assess the residual risks present to determine whether the gloves are suitable for their use. Attention; replace the gloves at least once every two hours.

- B. Precautions for use: 1. Always check the gloves for errors or imperfections before use. If the glove shows cracks or holes before or during use, discard them immediately. In case of doubt; discard them and put on new gloves. 2. Never wear gloves that are dirty on the inside or in combination with dirty hands, this causes irritation and can cause skin rashes. Only wear gloves on dry and clean hands. 3. When using the gloves while working with chemicals; - Ensure that the selected gloves are resistant to the chemical. Consult the chemical pictogram on the box and the detailed information later in this manual. - In any other case, they are allowed at most to be used against splashing or very short contact with chemicals. - In case of contamination, wash the gloves immediately with plenty of water before you take them off. - Ensure that no chemicals can penetrate the wrist. 4. DO NOT use these gloves as protection against mechanical hazards, heat, cold or against ionizing radiation. 5. These gloves are designed for single use, you must replace them regularly.

C. Composition / allergies: Some gloves may contain substances that can cause allergies to people who are particularly susceptible to this, resulting in irritations and / or contact allergic reactions. If suspecting allergy; contact your doctor or dermatologist. Please be aware; this product may contain (traces of) natural rubber / latex which can cause allergic reactions; consult the technical department of CLL B.V. if you require more information.

- D. Storage: 1. Store in original packaging in a cool and dry place, avoid exposure to excessive heat (>40°C) and / or moisture. 2. Keep away from sources of ozone, heat and open fire. 3. Open box should be shielded for exposure to direct sun or fluorescent lighting. 4. Do not use these gloves after the expiry date (see the reverse of the package).



E. Disposal (waste): During use, the gloves can become contaminated with contaminants or other hazardous materials. Reuse (after cleaning) is not possible. Respect the local regulations during storage and processing of disposed gloves.

F. Guarantee and warranty limitation: CLL B.V. guarantees the conformity of this product with the technical standard data of CLL B.V. on the date of delivery to the customers. Except insofar as prohibited by law, the present warranty is delivered to replace any other warranty, including any warranties for suitability for a particular use; the responsibility of CLL B.V. is limited to reimbursing the costs price of the product in question. The buyers and users of the product accept these warranty conditions, which cannot be changed by any other agreement, both orally and in writing.

G. Explanation of the icons:

Requirements (category of risks, sizing, marking, labeling, etc.). Includes CE mark, EN ISO 374-1:2016 / Type C, and EN ISO 374-5:2016. Contains a table for 'The gloves are tested with the following chemicals' and another table for 'Additional explanation to EN ISO 374-1:2016' with columns for Code, Chemical product, and CAS no.

H. More information: Do you need help to determine which glove is most suitable for your task? Ask our specialists via info@carellurvink.nl or call via 0031 (0)53 - 434 4343. Are you curious about the other variants of our certified work gloves or our complete range of products? Go to our webshop via www.carellurvink.nl or visit one of our servicecenters in Enschede (NL), Hilversum (NL) or Winschoten (NL).

Everyone deserves a Clean & safe environment



CLL B.V. IJzersteden 11 7547 TB Enschede Nederland

+31 (0) 53 434 43 43 +31 (0) 53 433 71 05 info@carellurvink.nl chat via carellurvink.nl

KVK 06063324 BTW NL800478253B01 IBAN NL80ABNA0445053259 BIC ABNANL2A

896M // vs. 23-10-20