




Reusable Face Mask



DGA tested



Reusable

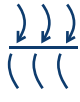

95% Filtration



OEKO-TEX® 100 Evolon® by Freudenberg



Light weight 5 gram



Machine Washing



Steam ironing



AFNOR SPEC S76-001 NBN/DTD S65-001



- Perfect fit, adjustable nose-wire & elastic strap
- Double layer, high quality breathable certified textile
- Ultra high protection, no filter required
- Light & comfortable, 5 gram weight
- Reusable & washable – minimum 50x
- Latex-free, hypo-allergy friendly & chemical-free
- Handmade in Europe by conTeyor

Technical specifications conTeyor Face Mask

- One-size-fits-all adult; children size available
- Wrap around face shield by 3 stitched folds
- Complies with AFNOR SPEC S76-001 & NBN/DTD S65-001
- OEKO-TEX® 100 Class 1 certified textile, Evo 40 PK, Evolon® by Freudenberg
- Non-woven, microfilaments polyester/polyamide
- Latex-free, hypo-allergy friendly & free of chemicals
- High breathability:
 - Air permeability at 100 Pa = 162 l/m²/sec
 - Breathing resistance < 40 Pa/cm²
- 95,1% protection against aerosols of 3µm particle size
- Dimensions: 205 mm (ear to ear) x 215 mm (nose to chin)

Why conTeyor?

Our core business is sustainable packaging, so no waste!

We decided to convert a part of our textile production for the manufacturing of masks and visors.

We are proud to be part of the global battle against COVID-19 and to support our communities.

Important user information

1. Recommended for non-medical, personal use
2. For hygienic purposes: change mask every day

How to use and put on the mask correctly

1. Wash and disinfect your hands thoroughly before using the mask
2. Cover mouth and nose with the mask and position elastic ear straps behind the ears
3. The mask has a nose clip, place it on your face in the nasal area and tighten to fit the shape of your face
4. Make sure that there are no gaps between the face and mask
5. To remove the mask, grab it by the elastic straps without touching front of the mask
6. After removing the mask, wash and disinfect your hands
7. Avoid touching the mask while using it, if you do, wash or disinfect your hands thoroughly

Care instructions

1. Machine wash at 60°C with a conventional detergent; preferably put the mask in a separate laundry bag
2. Air or mechanical drying, always followed by steam ironing at a temperature of 130°C

After the first washes, it is possible to see “pilling” appear on the fabric; this has no effect on the filtration quality of the mask



Product information conteyor Face Mask EU – Eng – v3

Disease prevention



DISCLAIMER: This product was designed as a mouth & nose cover and is not designed to be used by healthcare professionals. This product is not a 'personal protective equipment' (PPE) as defined in Regulation (EU) 2016/425 of the European Parliament and of the Council or 'medical device' as defined in Regulation (EU) 2017/745 of the European Parliament and of the Council. The use of this product is for non-medical purposes only and strictly at the individual's own risk. This product is not suitable for children under 3 years. The level of effectiveness depends on its use and circumstances. Considering the urgent need on short notice globally, the development process did not allow for extensive field testing and certification. The manufacturer assumes no liability for the use or misuse of this product.